

STEP 1: COLLECTION



SWISH & GARGLE
saline solution for
30 seconds.



SPIT
into funneled
collection tube.



POUR
blue cap
preservative
into funneled
collection tube.



SEAL
COLLECTION
TUBE with
red cap and
discard funnel.

STEP 2: PRINT & LABEL

PRINT "First Name" and "Last Name" on **PATIENT LABEL**. Affix lengthwise onto collection tube.

NOTE: DO NOT COVER BARCODE



STEP 3: ORDER

CREATE ONLINE ORDER by logging in to oraldna.com and selecting "Create Test Order(s)."

STEP 4: PACK & SHIP

PACK BOX with bagged collection tube; remove adhesive liner; close and seal box; **SHIP**.



PATIENT LABEL FOR COLLECTION TUBE
FOLD & TEAR HERE. ALLOW INK TO DRY.

First Name

Last Name

For shipping information and more visit: oraldna.com/kits